

Moving machine YWJ-01A



Moving machine YWJ - 01A



1. Product information

Name: Moving machine

Model No.: MKX-YWJ-01A

Max. load: 120kgs

Dimension: L: 700mm

W: 470mm

H: 940mm

Weight: 18kgs

Braking system: foot brake



2. Product features

a) Patented;

b) Assisting the mobility-impaired in moving from wheelchair to sofa, bed, bathroom and other places so that they can do washing, showering and treating on their own.

c) Wide-ranged folding design saves labour and reduces waist-bearing.

d) Max. load of 120kgs makes it applicable to different body shapes.


e) Adjustable height;

f) The mute wheels, wheel brake system and double buckles ensure a high security in using.

g) Water-proof materials;

h) Soft cushion for long-time sitting and various using applications;

- i) There is a small tube for infusion support. The tube is detachable and changeable to left or right side.
- j) Stylish colors apply to all kinds of scenarios.


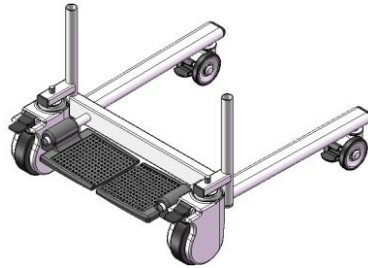


3. Product applications

This moving chair is delicately designed for patients and the elderly with mobility difficulty, and it is an ideal tool for families, medical institutions and nursing homes etc.


4. Assembly instruction

- a) Install two 5-inch wheels by both sides in front of the bottom holder, and two 3-inch wheels by both sides in the rear of the bottom holder. The operation is illustrated below:

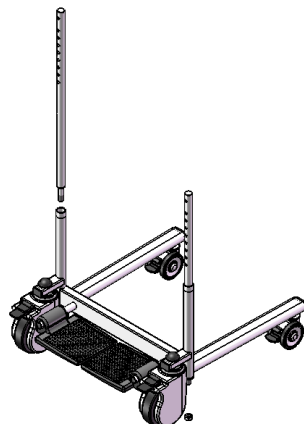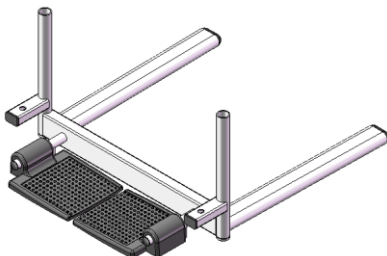

Note: Please press the brakes on the 3-inch wheels before assembly and disassembly.

b) Install the washer, M12-nut and plastic cover, as illustrated below:

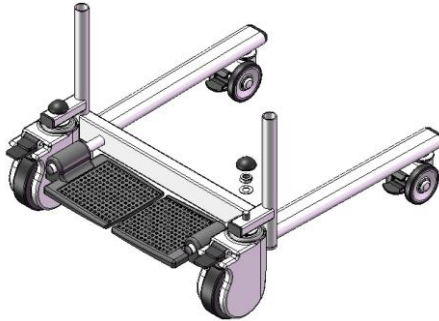


Note: The nut on the top of the 5-inch wheels (not omni-dimensional) must be fitted and fixed into the correct position on the bottom holder.

c) Place the rotatory shafts into the tubes before screwing the M12 nut. Make sure the shafts can be flexibly rotate without being loose, as illustrated on the right:


d) Place the handle part into the right shaft, as illustrated on the right:




e) Place the handle part into the left shaft, as illustrated on the right:

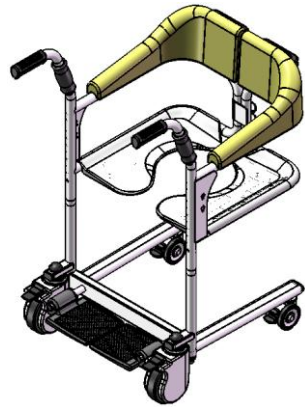


f) Adjust its height by rotating the handle to fit their hole sites into that of the rotatory shafts, then plug into latches. This step is illustrated on the right:

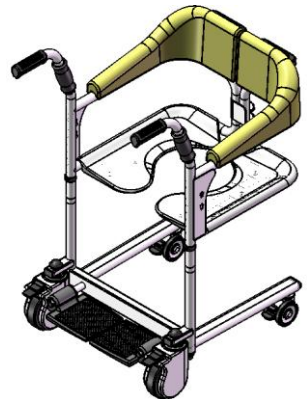

g) For placing a bedpan, fold the handles on both sides and place the bedpan holder into the hole site, then place the bedpan from the rear, as illustrated on the right:

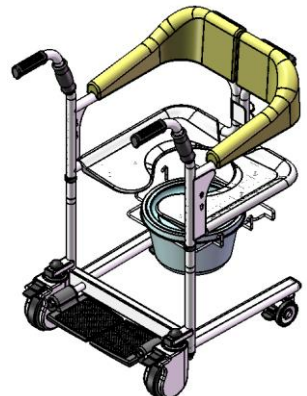

h) Fit on the small tube for infusion support. The tube is detachable and changeable to left or right side. As illustrated on the right:

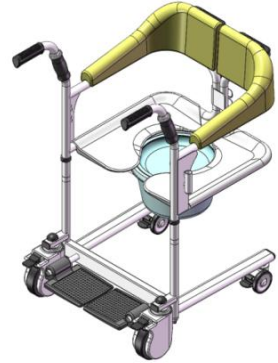

5. User manual

*** Please follow all the instructions stated in this brochure when using the moving machine. This is for your security.**


A. Instructions of moving the patient

a) The patient should sit up straight when the moving machine is unfolded in front of the patient;



Lift the patient's feet to step on the rest pedal. Make sure its brakes are locked before the patient is on the machine

b) When the patient is leaning to the left and to the right, turn the correlative side of the chair to fit the patient into it

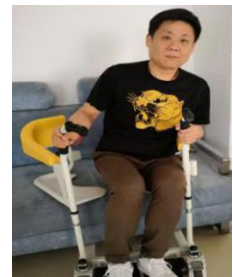

After the patient is seated, fold the machine from its back, and lock the auto buckle;



Then lock the safety buckle.

c) Unlock the brakes when the patient is seated with double buckles locked.

Now take hold of the front handle to move the patient.



d) Unlock the double buckles and unfold the machine after the patient is moved to the place.



B. Adjusting the height

a) Take down the lock catch to adjust the height. There are regulation ports on the sticks.

b) After adjusted to desirable height, put the lock catch pback to its lace.