Flexima® 3S / Softima® 3S

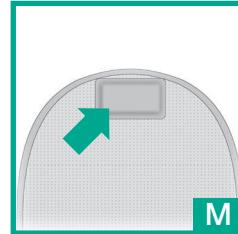
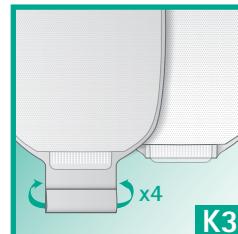
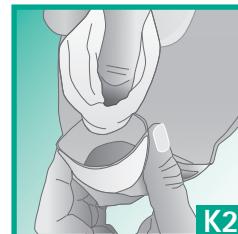
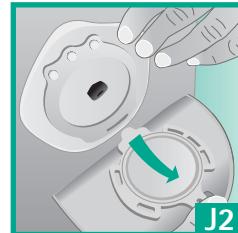
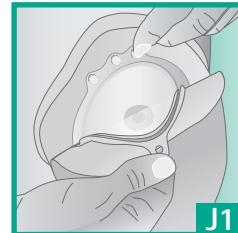
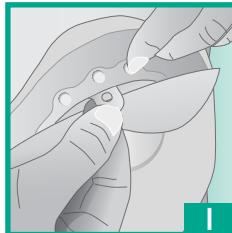
- EN** Instructions for use
- DE** Gebrauchsanweisung
- ES** Instrucciones de uso
- FR** Mode d'emploi
- IT** Istruzioni per l'uso
- NL** Gebruiksaanwijzing
- BG** Инструкции за употреба
- CS** Pokyny pro použití
- DA** Brugsvejledning
- FI** Käyttöohje
- HU** Használati útmutató
- JA** ご使用方法
- PL** Instrukcja użycia
- PT** Instruções de utilização
- RO** Instrucțiuni de utilizare
- RU** Инструкция по применению
- SK** Pokyny na použitie
- SL** Navodila za uporabo
- SR** Uputstvo za upotrebu
- SV** Bruksanvisning

B|BRAUN

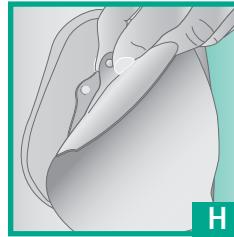
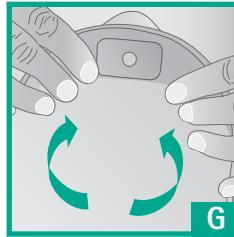
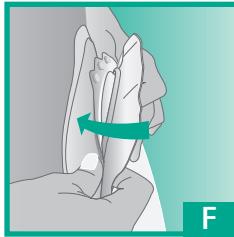
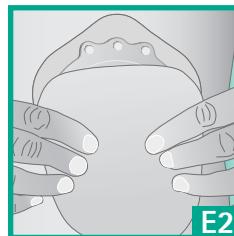
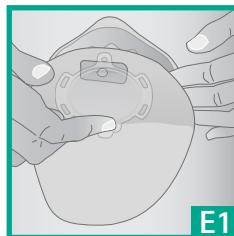
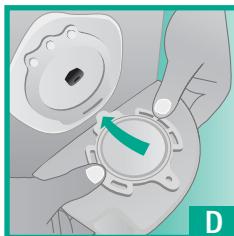
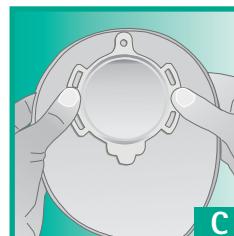
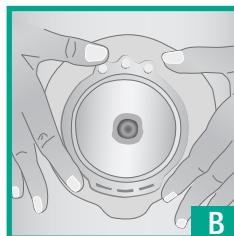
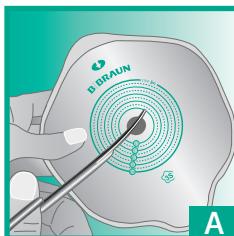
Flexima® 3S / Softima® 3S

Flexima® 3S / Softima® 3S

Flexima® 3S / Softima® 3S

Notes

Notes

EN Instructions for Use

The 2-piece collection system for enterostomy or urostomy patients is made up of:

- A base plate – flat or convex – with a skin barrier and a coupling ring for attaching the pouch;

■ Different types of pouches:

- > Closed;
- > Drainable with an integrated "Roll'Up" soft clamp system or a drainable High Flow cap, connectable to a remote collection pouch;
- > Drainable (for urostomies) with a drainage tab connectable to an overnight pouch.

The innovative semi-soft coupling set-up guide increases ease of use.

① Before placing the base plate

- Cut the skin barrier to the size of your stoma (**A**) using the measuring guide, remaining within the maximum possible cut-out and always allowing 2 to 3 mm around your stoma: it should not be strangled by the base.

② Preparing the skin

- Gently clean the peristomal area with warm water. Ether, alcohol and other antiseptics may prevent the base from staying in place, and should therefore be avoided.
- Carefully dab dry.

③ Placing the base plate

- Remove the protective cover.
- Place the base plate around your stoma taking care to place the 3 buttons at the top (**B**).
- Carefully press the skin barrier onto the skin.

④ Placing the pouch (closed & drainable)

- Check that the outlet of uro pouches is closed.
- Separate the internal sides of the pouch by creasing them lightly to allow some air in;
 - Hold the pouch by its side wings (**C**).
 - Insert the guiding tab, (situated at the bottom of the pouch's coupling ring) into one of the slots at the bottom of the base ring (depending on which way up you want to use the product) (**D**).

- Starting at the bottom of the rings, interlock the two rings by pressing them together progressing from bottom to top (**E1, E2**) over the entire circumference (**F**). It is possible to place the thumbs under the base plate ring to obtain a better grip and so avoid pressing against the abdomen; this must be done gently to avoid accidentally perforating the film under the base's coupling ring (**G**).

- Finalise the coupling of the two rings.
- Clip the pouch's upper tab on the corresponding base button (**H**).

⑤ Removing the pouch

Unclip the pouch's upper tab (**I**).

Separate the two rings by pulling gently on the pouch's upper tab in a downward motion (J1, J2).

- If necessary, clean the base plate with a non-shedding cloth or towel.

⑥ Using adhesive obturators on the filter

Before bathing or showering, hide the filter's gas evacuation vent.

To do this, stick an adhesive patch on the openings (M).

> The openings are situated on the same side of the pouch.

⑦ Drainable pouches

Drainable pouch with integrated Roll'Up clamp:

- Closing the drainage sleeve:
> Fold the sleeve four times using the guide and fold the tab back.
> To seal the pouch, apply pressure to the two self-fastening strips facing each other.
- Emptying the pouch:
> Position yourself to carry out the drainage, release the two self-fastening strips and unroll the sleeve. Be careful, the drainage will start at this point. To fully open the outlet place your fingers on each side (K1).

> Once the drainage is complete, expel, as much as possible, any matter still in the sleeve by pressing it lightly from top to bottom. Carefully clean its extremity (be careful: use non-shedding material) (K2).

> Then close the pouch.

Please note: the closed sleeve is hidden under the non woven (K3).

Drainable High Flow pouches:

- Use the cap provided in the box to close the pouch.
- To empty your pouch, remove the cap and point the pouch's opening towards the toilet bowl. Once the drainage is complete, clean with toilet paper and close with the cap.
- If necessary, the pouch can be connected to a higher capacity collection pouch (Flow Collector).

⑧ Urostomy pouches

Handling the drainage tap:

- Point the drainage system towards the toilet bowl, slowly turn the cap anti-clockwise until the desired flow rate is obtained (L1).
- To close the cap once the pouch has been emptied, turn clockwise until it has returned to its original position (L2).

Connecting to an overnight or leg pouch:

- Connect the pouch cap to the overnight or leg pouch using the supplied connector.
- Open the tap on the emptiable pouch to allow the urines to flow. Close the tap before disconnecting from the collection pouch.

Please note: If you would like to use a belt, ensure you use two diametrically opposed slots.

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- DK** B. Braun Medical A/S
- FI** B. Braun Medical Oy
B. Braun Medical SAS | 204 av. du Maréchal Juin | 92100 Boulogne Billancourt | France
- JP** Muranaka Medical Instruments CO.,LTD.
- SE** B. Braun Medical AB