## Flexima® 3S

Simplicity in motion.



Latest generation of 2-piece mechanical coupling system.



# Flexima® 3S: as simple as it gets!

As part of the latest generation of coupling systems, Flexima® 3S is:

#### **Simple:** it guides into position for you!

- Right positioning from the start: 3 easy fitting steps
- Insert the pouch tab into the base plate slot
- 2 Clip the bottom of the ring and slide it up on the other
- 3 Lock the pinpoint to secure the system
- No education required: intuitive guiding appliance
- **Multiple orientation:** 3 possible pouch positions according to the body configuration



The guiding system will completely revolutionize the patient's life: the insertion of the guiding tab into the slot is very intuitive.

No need to think twice!

Enteros to ma The rap is t





#### Secure: it gives you confidence!

- **Enhanced leak proofing** thanks to the close fitting mechanical coupling system
- **Extra security feeling** strengthened by an audible click and a final locking system
- Optional belt fixation: 3 hooks on the pouch add stability when needed






The system is extremely safe. It's easy to fit, no leakage between the baseplate and the pouch. The possibility to use a belt is very handy.

French Patient



- Minimum pressure on the abdomen: the floating flange prevents from discomfort at application
- High degree of comfort: flexible when base plate and pouch are coupled together
- Optimized body fit: new generation of plastic material allowing great discretion



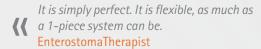


## Flexima® 3S: security in motion.



# A secure convex base plate for flush or retracted stomas

- Gentle convex slope for an optimum pressure display
- 6 mm depth together with a large base for an efficient protrusion
- Floating flange to prevent any discomfort at application


#### 3S BASE PLATE CROSS-SECTION VIEW

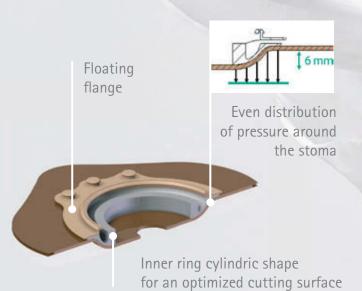





Flexima® 3S Convex base plate has been designed to capitalize on the benefits of Flexima® Key and mechanical 2-piece coupling systems: it ensures enough flexibility while providing a sufficient level of pressure on the peristomal area.

#### PRESSURE MAPPING OF 2-PIECE BBRAUN BASE PLATES

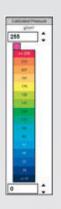

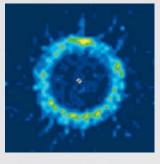



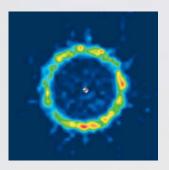

Flexima® 3S convex

Source: R&D data on file Test Report CR0663 (May 2014)

Whatever the choice, you are safe with B.Braun convexity.



## A secure urostomy outlet

#### **OUTLET FOR DAY**

- Clear position open/closed
- **Snaps perfectly** strengthened by an audible click and a final locking system
- Easy to handle: no risk of wet fingers







Open



- Audible click at connection
- Easy to connect to a night drainage bag
- Anti-twist design



Anti-twist connector



Direct connection to a drainage bag



### Flexima® 3S Uro

92% of uro patients rated Flexima 3S Uro\* "good" or "very good" on absence of leakage

\*Field Acceptance Test on panel of uro patients, France – September 2012 to January 2013

### A complete offer...

- 4 types of pouch: closed, drainable, high-flow and uro
- 2 types of base plate: flat and convex



### ... with unique features

- Leak proof filter on all enterostomy pouches
- Large 7cm Roll'Up outlet with easy opening and cleaning
- Optimised **pouch capacity** and **size** (compact shape)

## Flexima® 3S



Flat base plates (box of 5 or 10)		Diameter (mm)	References
	*	45/ 15-30	936415
		45/ 20	936420
		45/ 25	936425
		45/ 30	936430
	*	55/ 15-40	936515
		55/ 35	936535
		55/ 40	936540
	*	65/ 15-50	936615
		65/ 45	936645
	*	80/ 15-65	936815



Convex base plates (box of 5)		iameter nm)	References
Q	≪ 4!	5/ 15-26	937415
	4!	5/ 20	937420
	4!	5/ 25	937425
0	≪ 5!	5/ 15-36	937515
	5!	5/ 30	937530
	5!	5/ 35	937535
0	≪ 6!	5/ 15-46	937615
	6	5/ 40	937640
	6	5/ 45	937645

Accessories		References
BBraun Stoma Care Belt		5028
Flow Collector (2l bag)	(box of 5)	039901
Flow Collector (21 bag)	(box of 15)	039900

Closed pouches (box of 30)	Diameter (mm)	References beige	References transparent
Mini	45	931245	-
	55	931255	-
Midi	45	931045	O 931145
	55	931055	O 931155
	65	931065	O 931165
Maxi	45	931445	O 931545
	55	• 931455	O 931555
	65	931465	O 931565
	80	• 931480	O 931580

Roll'Up pouches (box of 30)	Diameter (mm)	References beige	References transparent
Mini	45	932245	-
	55	932255	-
Midi	45	932045	O 932145
	55	932055	O 932155
	65	932065	O 932165
Maxi	45	932445	O 932545
	55	932455	O 932555
	65	932465	O 932565
	80	932480	○ 932580

riigii-riow pouciics	Diameter	Hererences	Herefelices
(box of 30)		beige	transparent
	55	933055	O 933155
	65	• 933065	O 933165
	80	• 933080	O 933180

Urostomy pouches (box of 20 or 30)	Diameter (mm)	References beige	References transparent
	45	934045	O 934145
	55	934055	O 934155
	65	934065	O 934165



















Depending on your country, the brand name could be Flexima® or Softima® For each country, a specific letter is added at the end of the radical reference.

This document, its contents, including institutional data, information, trademarks and logos mentioned herein are the exclusive property of B.Braun. Any representation and / or reproduction, total or partial, of this document and its contents without the express prior consent of B. Braun, is strictly prohibited and vonstitutes an infringement of the intellectual property rights of B. Braun.

References: Field Acceptance Test, performed in France June 2012-Feb 2013 over a population of ostomy patients (22 colo, 17 ileo, 13 uro and 10 high flow bags) / Instructions For Use / Lab tests from R&D.

