


++++ Change of  ++++ Please read **marked** text carefully! ++++

IMTEC-oxLDL-ANTIBODIES Ig(GM)

oxLDL

ELISA for the Quantitative Determination of Anti-oxLDL Antibodies (IgGM)

Package Size

[REF] ITC59500 96 Tests Complete Testkit

[IVD]

Please read the instructions carefully before testing.

Procedural precautions:

Do not use the reagents beyond the date of expiry.

[DIL] DB16, **[WASH]** **[20x]** WB03, **[SUB]** TMB ELISA and **[STOP]** STOP ELISA may be interchanged between lots and test kits that share the same reagent designation.

All other reagents are specific for the individual test kit lot and must not be interchanged with other lots and test kits.

Store reagents at 2...8°C.

Intended Use

IMTEC-oxLDL-Antibodies Ig(GM) is an enzyme immunoassay (ELISA) for the quantitative measurement of IgG and IgM class autoantibodies against oxidized LDL in human serum and plasma. The assay is intended for professional in vitro diagnostic use only as an aid in the risk assessment for atherosclerosis and the antiphospholipid syndrome (APS).

Autoantibodies against oxLDL are used as markers of atherosclerosis, vascular diseases, and myocardial infarction. The IgG and IgM isotypes have the greatest clinical significance. Elevated anti-oxLDL antibody concentrations are found in case of atherosclerosis, myocardial infarction, hypertension, diabetes, renal failure/dialysis, SLE, preeclampsia, endometriosis and cigarette smoking. In contrast to other methods, the innovative and patent pending IMTEC-ELISA exposes each serum sample to both oxLDL and native LDL in one microtiter well. The uncomplicated analytical procedure permits the synchronous determination of IgG and IgM antibodies.

Principle

The test is based on the simultaneous incubation (patent pending) of serum samples with both oxLDL (immobilised on microtiter strips) and the native LDL (immobilised on pins of the TSP-plate). Subsequent binding of anti-oxLDL-antibodies from patient serum to the microtiter plate, is detected with a peroxidase-labelled secondary antibody that is directed against human IgG and IgM. After addition of substrate solution, a colour appears which intensity is proportional to the concentration and/or the avidity of the detected antibodies. Following the addition of stop solution, the colour switches from blue to yellow.

Reagents and Contents

[MTP]	1	Microtiter Plate ready for use, coated with oxLDL
[TSP]	1	TSP Plate ready for use, coated with native LDL
[CAL]	1 – 5 5 x 1.5 ml	Calibrators IgGM (white cap), human serum, inked according to concentration, ready for use Level of anti-oxLDL: 12.5 U/ml (1) , 25 U/ml (2) , 50 U/ml (3) , 100 U/ml (4) , 200 U/ml (5)
[NC]	1.5 ml	Negative Control Serum (green cap), human, ready for use
[PC]	1.5 ml	Positive Control Serum (red cap), human, ready for use Concentrations are stated on the labels.
[WASH] [20x] WB03	50 ml	Washing Buffer (black cap) Concentrate (20x) for 1 l TRIS buffer
[DIL] DB16	100 ml	Dilution Buffer (blue cap) ready for use Phosphate buffer
[CON]	15 ml	Conjugate Solution (white cap) anti-human-IgGM HRP conjugate, ready for use

[SUB] TMB ELISA	15 ml	TMB solution (black cap) ready for use, colourless to bluish 3,3', 5,5'-tetramethylbenzidin Hydrogen peroxide	1.2 mmol/l 3 mmol/l
[STOP] STOP ELISA	15 ml	Stop Solution (red cap) Sulphuric acid, ready for use	0.5 mol/l
	1	Adhesive Strip	

Safety Notes

Do not swallow the reagents. Avoid contact with eyes, skin and mucous membranes. All patient specimens and controls should be handled as potentially infectious. The controls have been checked on donor level for HCV and HIV-1/2 antibodies and HBsAg and found negative. Wear protective clothing and disposable gloves according to Good Laboratory Practices.

All materials contaminated with patient specimens or controls should be inactivated by validated procedures (autoclaving or chemical treatment) in accordance with applicable regulations.

Stability

The reagents are stable up to the stated expiry dates on the individual labels when stored at 2...8°C. After opening the reagents are stable for 4 weeks.

Reagent Preparation

Allow the testkit and all its components to reach room temperature before use! Used bottles should be closed carefully and stored at 2...8°C. Store **[SUB]** protected from light.

Do not use polystyrene vessels for handling of **[CON]**.

To avoid potential microbial and/or chemical contamination, unused reagents should never be transferred into the original vials.

Washing Buffer Solution **[WASH]**

Any crystallised salt inside the bottle must be resolved before use. Dilute 1 part **[WASH]** **[20x]** with 19 parts distilled water. **[WASH]** is stable for 6 weeks stored at 2...8°C.

Specimen

Patient sera

Use samples freshly collected or freeze samples at –20°C. **Freeze and thaw once only.** Do not use serum samples inactivated by heat treatment at 56°C.

Allow the samples to reach room temperature (30 min.).

Dilute sera 1:101 with **[DIL]** (add 10 µl serum to 1 ml **[DIL]**).

Procedure

Do not touch the coated side of **[TSP] with finger tips.**

- **Pipette 100 µl** diluted sample, **[CAL]**, **[PC]** and **[NC]** into **[MTP]**, for blank use **[DIL]** instead of sample dilution, seal **[MTP]** with **[TSP]**.
- Incubate for **1 hour** at RT.
- Mark used wells of **[MTP]** and used pins of **[TSP]**.
- Remove **[TSP]** with the pins upright., Let it dry for 1h at RT and store **[TSP]** in the lockable bag at 2...8°C.
- **Discard** the solution from **[MTP]**. **Wash** **[MTP]** 3 times using 300 µl **[WASH]** per well.
- **Discard** **[WASH]** and knock out residues on an absorbent paper or cloth.
- **Pipette 100 µl** **[CON]** and seal **[MTP]** with adhesive strip.
- Incubate for **30 min.** at RT.
- **Discard** the solution from **[MTP]**. **Wash** **[MTP]** 3 times using 300 µl **[WASH]** per well.
- **Discard** **[WASH]** and knock out residues on an absorbent paper or cloth.
- **Pipette 100 µl** **[SUB]** and incubate for **10 min.** At room temperatures above 25 °C the substrate incubation could be shortened, but should never fall short of 5 min..
- **Add 100 µl** **[STOP]** per well.
- **Read absorbance values at 450 nm** within the next 10 min. after stopping. Bi-chromatic measurement with a reference wavelength at 620 – 690 nm is recommended.
- Tap out **[MTP]** and **let dry for 1 h at RT** and store **[MTP]** in the lockable bag at 2...8°C.

Validation of the Test

The test results are valid provided the following criteria are met for the obtained results:

- **PC** is within the indicated range (see label).
- **NC** is lower than the cut-off-value of the test.
- **CAL|5** does not fall below an absorbance value of 0.6.
- The absorbances of **CAL|1-5** keep raising.

In order to improve accuracy of the test results we recommend to run **CAL|1-5**, **PC**, **NC** and patient samples in duplicate.

Interpretation of Results

Plot measured absorbencies against U/ml of **CAL|1-5** in semi-log. By interpolating the plotted measuring points, a calibration curve is obtained, from which the concentrations of anti-oxLDL antibodies in the patient samples can be determined.

Results above 30 U/ml (cut-off value) are positive.

Limitations

A positive result must be used in association with clinical evaluation and diagnostic procedures. The values obtained from this assay are intended to be an aid for risk assessment only.

Elevated anti-oxLDL antibodies may occur in individuals with no evidence of clinical disease.

If the patient sample contains elevated levels of immune complexes or other immunoglobulin aggregates, false positive results by non-specific binding cannot be ruled out.

Performance characteristics

Typical performance data can be found in the Verification Report, accessible via:

www.human.de/data/gb/vr/el-59500.pdf or

www.human-de.com/data/gb/vr/el-59500.pdf

If the performance data are not accessible via internet, they can be obtained free of charge from your local distributor.

Safety Notes

STOP Warning

• Hazard statements

H315 Causes skin irritation.

H319 Causes serious eye irritation.

SUB Danger

• Hazard statements

H360D May damage the unborn child.

• Precautionary statements

CAL **NC** **PC** **WASH|20x** **DIL** **CON** **SUB** **STOP**

P234 Keep only in original container.

P260 Do not breathe dust/fume/gas/mist/vapours/spray.

P262 Do not get in eyes, on skin, or on clothing.

P281 Use personal protective equipment as required.

P303+P361+P353 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower.

P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

P337+P313 If eye irritation persists: Get medical advice/attention.

P401 Store in accordance with local/regional/national/international regulations.

P501 Dispose of contents/container in accordance with local/regional/national/international regulations.

References

1. Conrad K. *et al.*, Autoantibodies in Systemic Autoimmune Diseases – A Diagnostic Reference; Pabst Science Publishers, Lengerich, Berlin, Riga, Rom, Viernheim, Wien, Zagreb (2002)
2. Becarevic M *et al.*, Clin. Lab. **51**, 279-283 (2005)

EL-59500

INF ITCS9500 GB

08-2018-15M



Human