



MEDTEC

MEDICAL DEVICES PRODUCT


CHATURONG COOLING LIMITED
PARTNERSHIP



WATERPROOF BANDAGE SPRAY

MEDTEC Waterproof Bandage Spray is a convenient and easy to use quick-dry spray for protecting small and superficial wounds. This formula provides transparent, breathable, flexible and waterproof film that immediately protects against water, dirt and bacteria. Do not use on heavily bleeding cuts.

Directions : Clean and dry the affected area before application. Shake well before use. Hold the can 10 cm away from the affected area. Spray a light coating on the affected area and let dry. Do not spray onto face. A stinging sensation may occur briefly upon application. The plaster will gradually disappear over time. For the removal, apply the alcohol onto the film and wipe off.

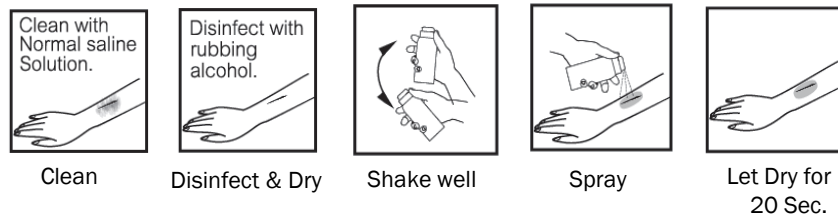

CAUTIONS:

- Do not use it on abrasions, infected wounds, or deep burns.
- Do not use on face.
- Avoid contact with eyes.
- Do not inhale.
- The application may cause a slight tingling sensation.



Sample pictures of wounds that can be sprayed with a Waterproof Bandage Spray



COMFORTING CAST SPRAY

Fresh and light Skin Feel


Cool & Relief Skin Discomfort

Eliminate Odor


Infused with Tea Tree Oil

Comforting Cast Spray is a suspended powder formulated especially for cast wearer. This formula will prevent skin discomfort by giving a cooling sensation, eliminating odor and displacing water inside the cast.


Directions : Shake well before use. Insert the tube into the nozzle. Insert the tube into an affected area and spray liberally. Can be used as often as needed or every 4-6 hours. Do not spray on an open wound.

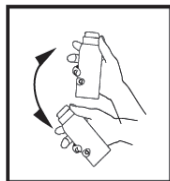


Shake well.

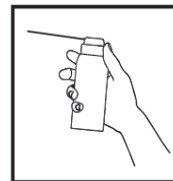

Insert the tube into the nozzle.

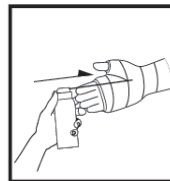

Insert the tube into the affected Area.

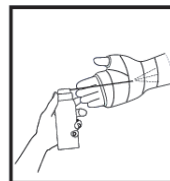

Spray.



ICY SPRAY

Muscle Pain Relief


For sprains and muscle aches

MEDTEC Icy Spray gives an instant blast of cold to relieve pain. It is ideal for sport first aid treatment. Spray around injuries such as sprains and aches. MEDTEC Icy Spray can be applied during exercise or immediately afterward.


Indications : Use as a first aid measure for muscle injuries. For relieving bruises and sprains.

Directions : Shake well before use. Hold the can 10-15 cm. away from the skin.


Continuously spray (no longer than 5 seconds) on the affected area. Frostbite may occur if the application is continuously sprayed for more than 5 seconds. The application should be repeated every 25-30 minutes within the first 24 hours after the injury. Do not spray on face and open wound.

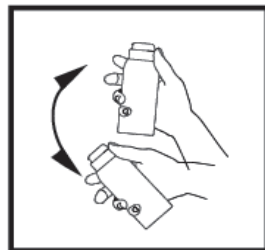

Shake well.

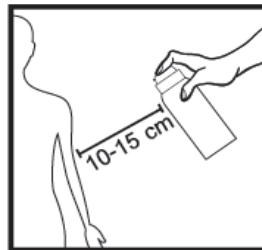

Hold the can
10-15 cm away
from the skin.

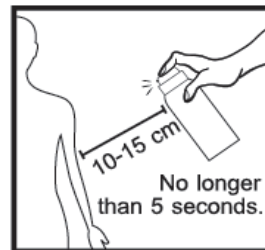

Spray for
No longer than
5 sec.




AIR OF LIFE PURE OXYGEN



Portable oxygen for those experiencing shortness of breath during sport, outdoor activities or any occasions that require immediate use of oxygen. MEDTEC Air of Life Pure Oxygen can be used for oxygen therapy.

Directions :

1. Remove the bottle cap and detach the inhalation cap from the bottle.
2. Attach the inhalation cap to the nozzle.
3. Place the inhalation cap over the nose and mouth. Press down the nozzle for 10-15 seconds while inhaling and release the nozzle while exhaling.
4. Repeat as needed. (8-10 times)

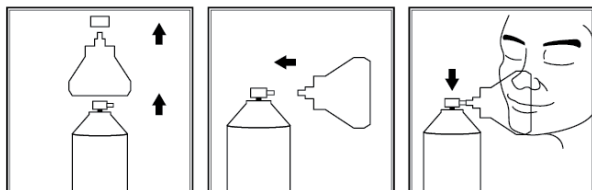

MICRO DIFFUSION NASAL SPRAY

An isotonic normal saline solution that is formulated with natural sea salt. This product use micro-diffusion technology that assists a soft and gentle application. It is recommended for children and adults for the daily cleansing of the nasal airway, particularly in case of dried or obstructed nose.

Indications: For daily hygiene of the nasal airway, for the humidification of the dried nasal mucous membrane, and as a compliment to the treatment of ENT disorders.

Directions: Tilt the head to one side, Insert the nozzle into the nostril. Hold the breath. Push the nozzle allowing the saline solution to flow Inside the nasal cavity. Tilt the head back into an upright position Allowing the substances to come out of the nostril. Wipe off the excess. Repeat the steps to the other side. After use, clean the nozzle with soapy Water. Rinse off with lukewarm water. Air-dry the nozzle.



COMING SOON

MEDTEC Stay Clean Hand Sanitizer

Alcohol 72.4% v/v

MEDTEC Stay Clean Hand Sanitizing Product is a hand sanitizer that composes of ethanol at 72.4% (v/v) concentration and isopropyl methylphenol (IPMP or o-cymen-5-ol) as the main active ingredients. This combination reduce the amount of germs causing disease. Highly efficient with excellent safety profile.

MEDTEC Stay Clean Hand Sanitizing Product is formulated to protect and nourish the skin simultaneously. The product is infused with Pentavitin® (Saccharide isomerate), a patented Swiss-made natural extract that continuously protect and tackle skin dryness. Suitable for frequent hand washing and excess use of alcohol-based hand sanitizer.

This product is lightly scented with allergen-free fragrance.





Contact Information :
Dr. Chaturong Charastrakul
Managing Director

- Chaturong Cooling Limited Partnership,
No.44/1-2 Moo 5, Phatum thani-Banglane Road,
Rat Niyom, Sai Noi , Nonthaburi, Thailand 11150
- Tel: +66 (0) 2 923 1102 , +66 (0) 2 923 1886
- Fax:+66 (0) 2 923 1887 Email : chaturong@chatcooling.com